

7.5 PRINTER INSTALLATION

7.5.1 Printer installation

Proceed as follows:

- Access to the configuration mode with level 2 minimum



1. Start the FibroScan®.
2. On the home page, press .



3. The window shown opposite gives you the choice of various login options. There are six, of which three are accessible by distributors and two by the customer.
 - Level 2**
Login: Biomedical engineer
Password: maintenance




Do not connect the printer at this stage.
Do not follow the instructions provided by the printer manufacturer.

- Go to "printer tab".




- Insert the printer CD driver installation
- Turn on the printer and connect it to one of the USB ports on the FibroScan®

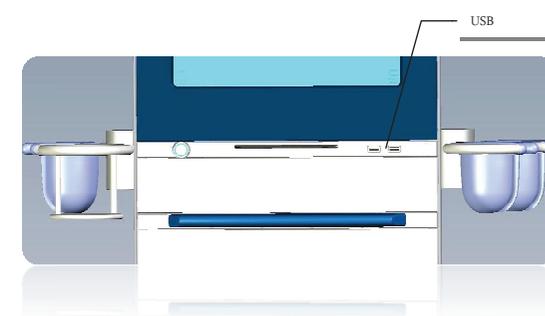

Warning: if the printer has been disconnected, make sure it is re-connected to the same USB port after turning on the FibroScan in order for it to be recognized by the operating system.

- Wait for some seconds. A window « Hardware Wizard window » is displayed; select « Not this time ». And click on "Next".
- If not then click on "add printer" and follow the instruction.
- The window « What do you want the Wizard to do? » is displayed; select « Install from a list or a specific location ». And click on "Next".
- A new window is displayed; on this window select « Search from removable media (floppy, CD-ROM) ». And click on "Next".
- Wait for some seconds. A window with a list of driver is displayed; select the appropriate driver and follow the instruction till the end.
- Click on "Printer Selection Tool" on the "User interface" tab.
 - If the printer can be used with the system, it will be displayed in the list

- If there are several printers in the list, ensure that the one you wish to use is the default printer.
- If this is not the case, right-click on the printer you wish to use and select "Set as Default Printer".
- Click on "OK".
- Turn the FibroScan® off and on again (switch in the lower left-hand corner of the monitor).

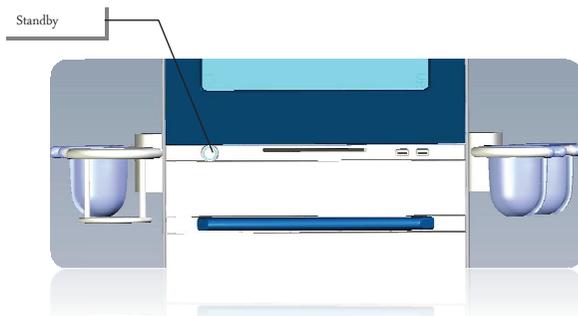

Warning: if the printer has been disconnected, make sure it is re-connected to the same USB port after turning the FibroScan® in order for it to be recognized by the operating system.



When running the first print job, a "Print error" message may be displayed. Click on "OK" to restart the print job.

8 MAINTENANCE

8.1 SPARE PARTS LIST

Zone	Reference	Designation
Probe	AP107_100	M Probe
	AP107_200	S Probe
	AP107_300	XL Probe
	AP107_007	Probe S, M, XL Case Foam
	AP107_008	Probe S, M, XL Case
Computer zone		
E300_001		FS502T Mother board Set
E300_002		FS502T Sata Hard disk drive prepared
E300_003		FS502T Sata DVD-ROM Drive
E300_004		FS502T Display set with Touchpad
E300_005		FS502T Screen Inverter board
E300_006		FS502T Toucpad Inverter board
E300_007		FS502T On/off button
E300_008		FS502T Sata power cable
E300_009		FS502T Sata HDD cable
E300_010		FS502T Sata DVD cable
E300_011		FS502T Toucpad Inverter board cable
E300_012		FS502T LVDS cable
E300_013		FS502T Mother board power cable
E300_014		FS502T Screen Inverter board CABLE
E300_015		FS502T DVI CABLE
Elastography engine zone		
AP40012		FS502T Elastography engine V2
Power supply zone		
E300_016		FS502T Power supply
E300_017		FS502T Main filter
E300_019		FS502T Remote control board
487_088		FS502T with brake
487_088		FS502T without brake