

# 1. SOFTWARE UPDATE

When the software needs updating (notification by ECHOSENS<sup>®</sup>), follow the steps below:

1. Start the FibroScan<sup>®</sup>.
2. Once the software has booted, insert the CD provided by ECHOSENS<sup>®</sup> into the CD-ROM drive and reboot the FibroScan<sup>®</sup>.

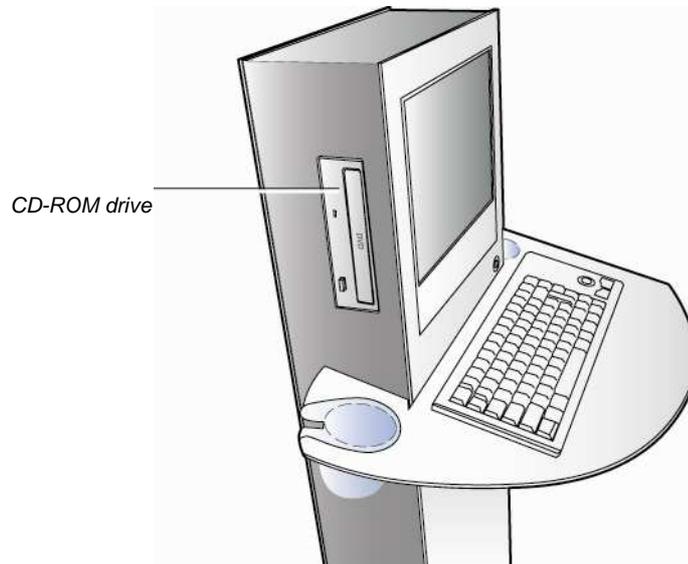


Figure 1: position of the CD-ROM drive.

3. When the system reboots, a warning message is displayed. Press the "Y" key and confirm by pressing "Enter". The update procedure starts.
4. Once the update is complete, the CD-ROM is ejected. Remove the CD-ROM and press "Enter" to reboot the FibroScan<sup>®</sup>.
5. The system is updated and the software starts automatically.



If a "Hardware error" message is displayed, do not acknowledge it until the following procedure is complete.

---

6. **AFTER THE FIRST BOOT, TURN OFF AND REBOOT THE FIBROSCAN<sup>®</sup>.**
7. Once the FibroScan<sup>®</sup> has rebooted, press "Ctrl+F12", release "F12" first, then type:
  - Login: Hospital.
  - Password: maintenance.



Input is case-sensitive.

---

8. You can now reconfigure the FibroScan<sup>®</sup> (Time, Date and Printer)
9. The update is complete.



Do not install FibroScan<sup>®</sup> software on another computer. This results in hard drive formatting.

---