

1. REQUIRED

Concerning this procedure, do you need those element :

- 402/430/530 (with or without probe)
- Electrical cordon
- A Echosens Software Flash Key


2. FIRST START UP

Insert the USB flash key at this moment. Start up your device normally


The device will start the restoring automatically (some black background and blue background will appears) depending on the device it might reboot itself or just shut down, remove the USB thumbdrive and power on the device

If you have any question about this instruction, please join us at service@echosens.com

FibroScan 402

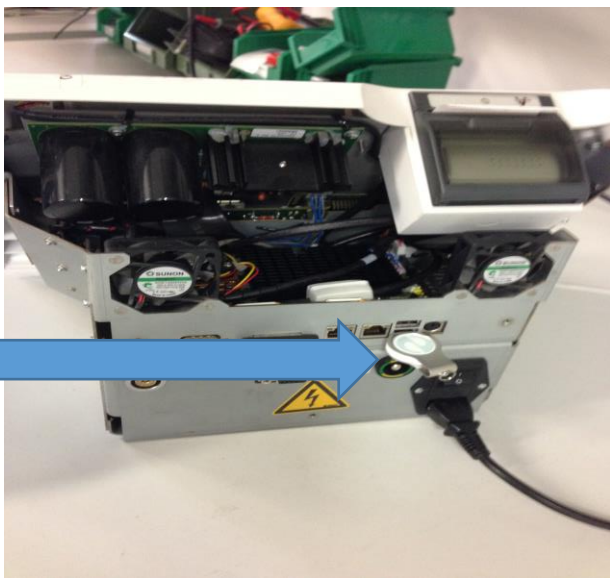

FibroScan 530

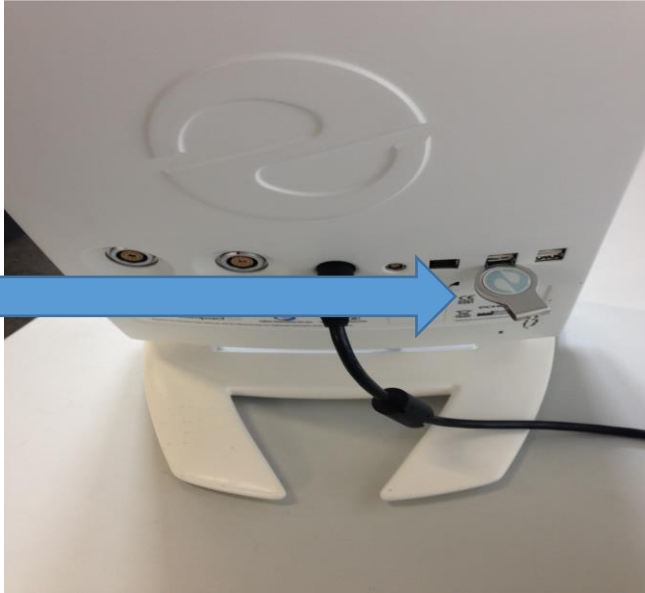

FibroScan 430

