

## BACKUP OF PATIENT DATA

To backup patient data of your FibroScan® equipped with a software version upper than **A2.0** for FibroScan® 502 or **C1.0** for FibroScan® 502 Touch, follow the steps below:

1. Unplug all USB devices from your FibroScan®.
2. Plug a USB mouse.
3. Start your FibroScan® and shortly insert the "Data Backup CD".
4. A command window and a pop-up are displayed.
5. Before confirmation to continue the process, plug an empty non-encrypted USB Hard Drive.
6. Confirm the continuation of the process.
7. Wait for the process to be complete (the copy takes about 5 min per GB).
8. A pop-up indicating if the process succeeded or failed, appears. Confirm the information. Restart the process in case of failure.
9. The CD is ejected and the FibroScan® is shutting down.



Some archived patient examinations (.fibx files) may be not importable. To review these examinations, please use FibroView Report, or contact Echosens or its local representative: [service@echosens.com](mailto:service@echosens.com).

---



For a given patient, in case of multiple examinations, importation may create as many patient files as examinations.

---