

Dear Sirs,

Next week we will install the Dicom option on the ISYS lithotripter. In order to save time, I would like to inform you that the ISYS AETitle will be: **SONOLITHISYS**.

We will need to set different parameters so that our computer can communicate with your server. That is to say:

To be detailed so that we can communicate with the server

The screenshot shows a 'Form' window with the following sections:

- Service Class User (SCU)**
 - ISys AETitle : SONOLITHISYS
- Store Service Class Provider (SCP)**
 - Server Name : OLBoxSStore
 - AETitle : SCP
 - IP Address : BMA-XP
 - Port : 8007
 - Test button
- Worklist Service Class Provider (SCP)**
 - Server Name : SWorkList
 - AETitle : BMAWKL1
 - IP Address : BMA-XP
 - Port : 3104
 - Test button
- Product Information**
 - Product family number : 1
 - Product type number : 2
 - Product serial number (1 to 999): 888

At the bottom are 'Save' and 'Cancel' buttons. A red box on the left contains the text 'To be detailed so that we can communicate with the server', with a red bracket pointing to the SCP sections.

We would appreciate very much if the parameters to be detailed (See above) could be available for the installation of the Dicom on our lithotripter, upon our arrival.

Finally a **worklist should be available** so that we can test ultimately the Dicom when the Dicom settings will be completed.

Best regards and looking forwards to meeting you soon,