

# Table Remote Light Indicator Meanings

This symbols means :

○ : Led off

● : Led lit

\* : Led flashes



A) Whilst requesting the **LOADING/ UNLOADING** position of the table, the alarms shown below will occur if the case condition is not met.

Case						Action
Case 1: Generator is not in parking position	*	○	○	○	*	Put generator in parking position
Case 2: Generator is not in high position	○	*	○	○	*	Put generator in high position
Case 3: C-Arm not vertical in X	○	○	*	○	*	Zero C-Arm
Case 4: C-Arm not vertical in Y	○	○	○	*	*	Zero C-Arm
Case 5: In LITHO mode and table shutter is still IN	○	○	*	*	*	Select ENDO mode on table remote control
Case 6: In ENDO mode on remote control	*	○	*	○	*	Move generator to LOW position

If many of the above cases are not satisfied, the lines of light conditions can be **combined**.

B) When the following lights appear, look at the necessary action to be taken.

Case						Action
Case 1: I-SYS Lithotripsy mode selected	*	○	○	○	●	Check II anti-collision
Case 2: Endo-Uro mode selected	○	○	*	○	●	Check II anti-collision
Case 3	*	○	*	○	*	Release E-stop button on side of table

C) If the table limit is reached in one or several directions, the light indicators below will flash.

Case					
Case 1: I-SYS Lithotripsy mode selected	*	○	○	○	*
Case 2: Endo-Uro mode selected	○	○	*	○	*