


| | | |
|--|----------------------------------|---|
| Service Info | : SI43764 | <input type="checkbox"/> Literature |
| Type | : 1202,2202,2300 | <input type="checkbox"/> HW |
| Concerning | : Log files without patient data | <input type="checkbox"/> SW |
| | | <input checked="" type="checkbox"/> Other |
| PRIORITY | | |
| <input checked="" type="checkbox"/> Info | | |

How to send the log files without patient data


Before sending the log files, the information of patients contained in the screen captures needs to be deleted. The script “DeletePatientInfo.bat” will run through all folders on the USB key from the root of the USB key and identify all .png files.

Important: All .png file on the USB key will be deleted!

1. Copy the script “DeletePatientInfo.bat” to a USB key
2. Get the log files from a scanner to the same USB
3. Open the USB (D:\) folder on a computer.

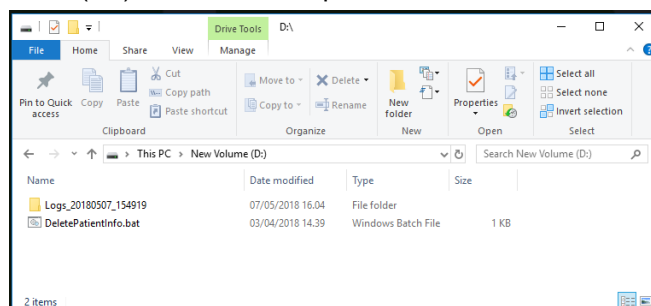

4. Click on “DeletePatientinfo.bat” and Run the batch file.

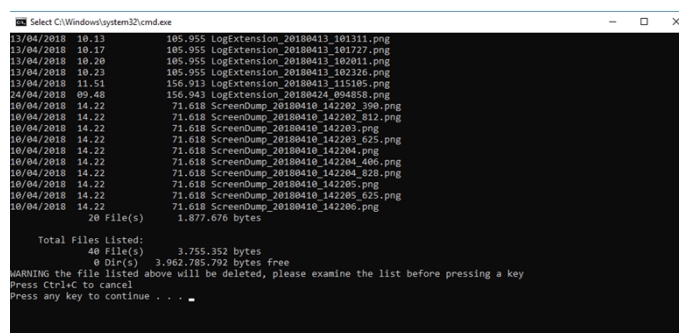

5. Examine the list and press any key to continue. Although if cancelation is needed press ctrl+c.
6. Zip the logfiles before sending them.