

X-RAY HF1 R/8 MONOBLOC

CODE: XRM.11.C22.001

CLASS: I B (IEC 601-1)

MONOBLOC: HF1 R/8

SERIAL NUMBER: / -

X-RAY INSERT: RO 0306 0.3 0.6

SERIAL NUMBER:


OUTPUT VOLTAGE FREQUENCY RIPPLE: 40 kHz

ROTATING ANODE: 3000 Rpm

SMALL FOCUS: 0.3 mm

LARGE FOCUS: 0.6 mm

FILAMENT POWER SUPPLY: 20 KHz

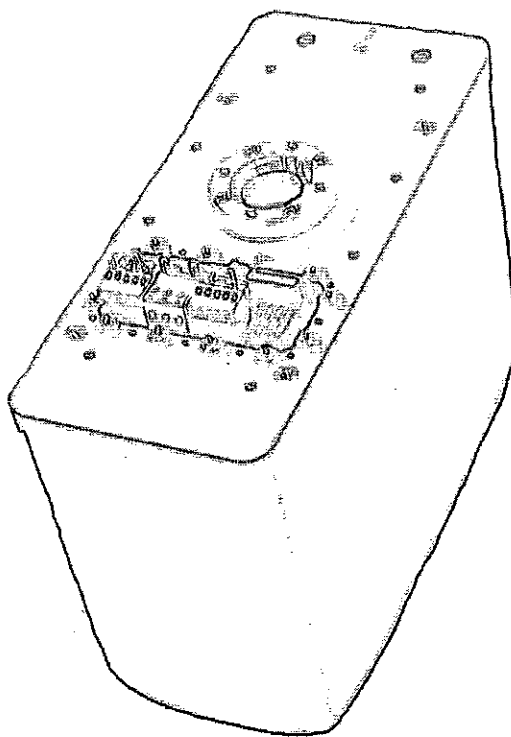

*May be
reconditioning.*

CE 0051

Medical Device Class IIB

NOTE - COMPLETE TEST REPORT ON DEMAND

WARNING:

IF THE EQUIPMENT HAS REMAINED INOPERATIVE FOR MORE THAN THREE MONTHS, PROCEED TO RESET THE TUBE AS FOLLOWS

RADIOGRAPHY / FLUOROSCOPY : 70 kV, 3 mA, FOR 5 min. + 5 min. PAUSE REPEAT 5 TIME

RADIOGRAPHY : 70 kV, 20 mA, FOR 0.5 sec. + 30 sec. PAUSE + RISE THEN UP TO kV_{MAX} AT STEPS OF 5 kV