

YSB-III Medical Shaver Systems

Operation manual (4th edition)

This product can only be used by professional medical personnel in the operating room. This manual only describes the use of the product, not the treatment manual of endoscopic surgery

1 Installation and power connection

The shaver system should be placed on a stable horizontal plane, and the power cord provided by the equipment should be connected to the main socket with protective grounding terminal; Before turning on the power switch of the network, check whether the power cord matches the power supply; After checking the power supply, the device should be connected to the preset potential equipotential terminal and fully grounded.




Caution


- Keep the air outlet of the motor fan unobstructed to ensure that there is no foreign matter blocking.
- If the main engine is placed on the trolley, the area of the trolley should be greater than or equal to the area of the bottom of the main engine.
- The power plug must be fully inserted into the socket, so as not to be pulled out during use and interrupt the operation.
- Please use the power cord provided by our company, do not refit the power cord.

2 Device connection

Connect the device with 2.5L tissue collection bottle (see the figure below)

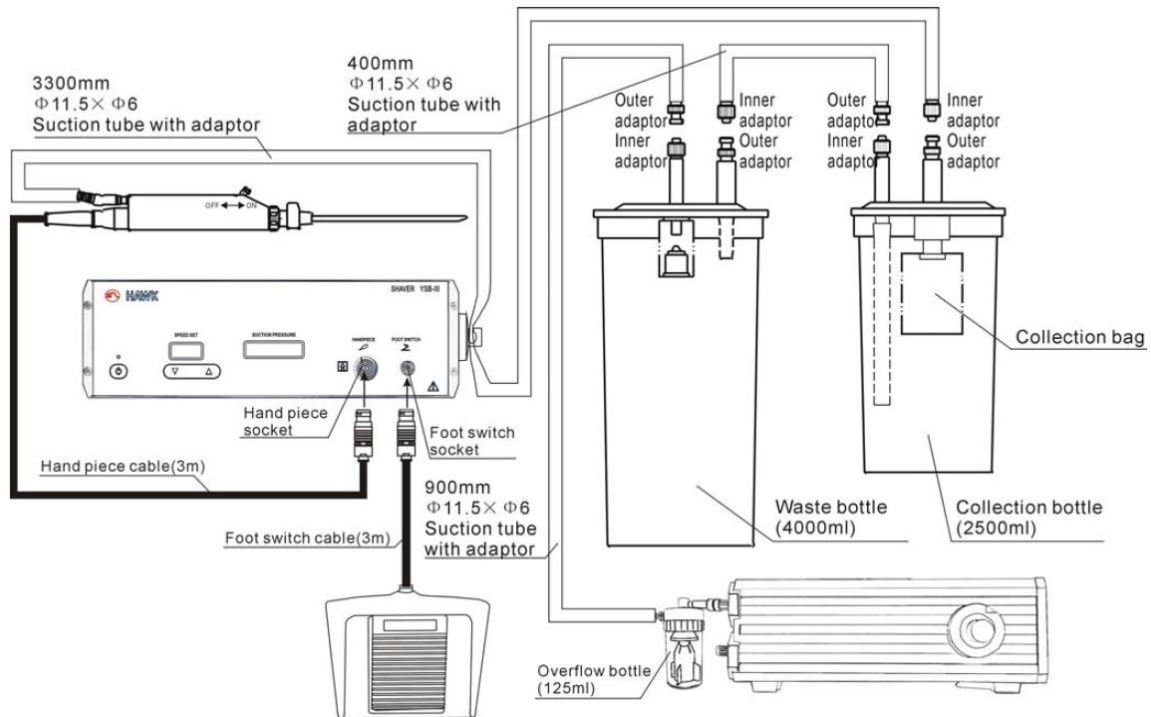

Connect the device with disposable external suction connecting tube (see the figure below)

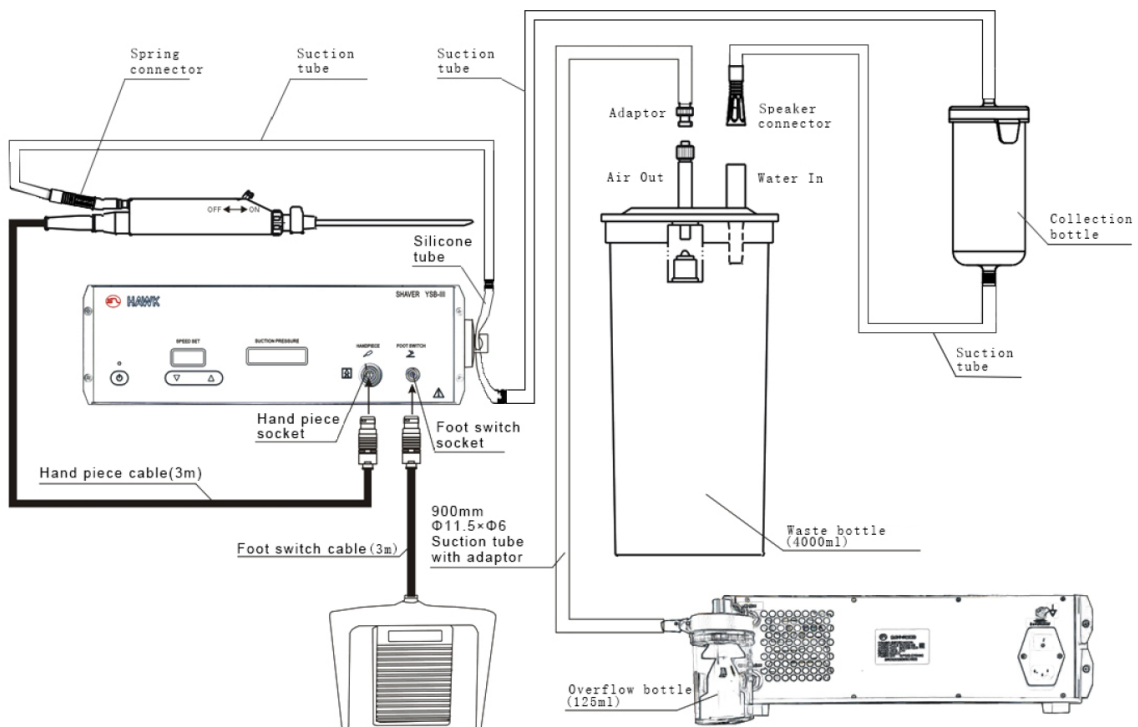

3. Maintenance

1. If there is something wrong with the device, check the power cord and other connecting wires first. Please search the following table before contacting the service provider. If the problem persists, shut down and unplug the power cord and contact your local technical service or designated hawk service technology department.

2. Frequent problems can be easily solved by checking the obvious and easily overlooked problems. If the problem still exists, check whether it can be solved according to the following procedures before consulting the service provider. Doing simple checks in advance can save you time and money.

Symptom	Possible Causes	Processing Method
The power indicator doesn't work	The plug and socket are not connected	Connect the plug and socket and turn on the switch
	The power switch is not on	Turn on the power switch
	The fuse is blown	Replace the fuse. If it burns out after installation, please check the voltage
Hand piece doesn't work	The plug of hand piece is not in place	Whether the plug of hand piece or foot switch is firmly connected with the socket on the controller and locked (Gently pull the back of the plug, it should not come out)
Abnormal operation, poor cutting	The tool is not fixed firmly	Removing and reloading
	The suction pipe is not smooth	Check the vacuum and suction channel
	The water inlet and outlet switch on the handle is in the off position	Push forward the handle of water inlet and outlet switch to open it

See more details in full user manual

HANGZHOU HAWK OPTICAL ELECTRIC INSTRUMENT CO., LTD.

01/27/2021