


HI-LO STRETCHER TROLLEY

(KK-725 • 726 • 727 • 728E) Parts list

This parts list contains the parts that are required for repairing and maintainig HI-LO STRETCHER TROLLEY.

- **Please note that the listed parts are the smallest unit that can be provided.**
- **Please read the instruction manual to double check the parts name and parts number.**
- **Due to improvement of the product, parts of this parts list may differ slightly from the product purchased.**

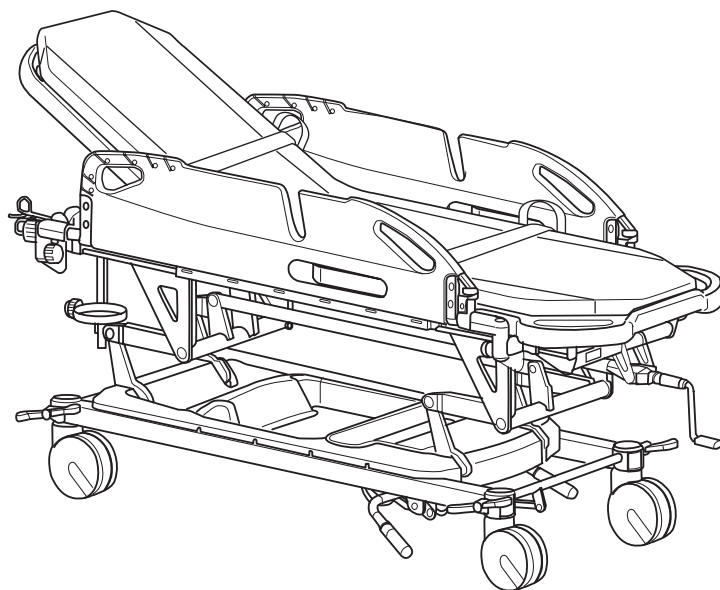

PARAMOUNT BED CO.,LTD.

HI-LO STRETCHER TROLLEY Parts list

Contents

	Page
1.MATTRESS BASE • SIDE RAIL (TRANSFER BOARD) • • • •	3-4
2.MAIN FRAME • HI-LO LINK • • • • • • • • • • • • • • • •	5-6
3.BASE FRAME •	7-8

How to use parts list

- The color or dimension differences do not reflect on the diagrams. Please check the part table for correct unit color and size.
- Each part is numbered. The number in picture and the number in chart corresponds to each other.
- Be sure to inform us the parts number for parts inquiry.

When ordering repair parts

- Each parts will be ordered by unit of one.
- When ordering, please inform us the parts number and quantitiy required.

Repair during the warranty period

While most repairs are carried out free of charge as specified in the warranty card, some types of repairs are carried out at the user's expense even during the warranty period.

See the warranty card for more information.

Do not lose the warranty card. As it would be at user's expense without it.

After the warranty

When problems can be solved by repairs, they can be carried out at the user's expense.

Minimum part availability period

Paramount Bed will stock bed replacement parts for repair (needed to maintain product performance) for at least 6 years after production has been discontinued.

Cautions


- For user's convenience, Paramount Bed try to provide the parts in details as much as possible. However due to safety and quality aspect and efficiency for delivery, some parts may be delivered in assembled form.
- Paramount Bed don't provide packing materials (cardboard, tape, band, etc). Please prepare them at customer's cost.
- Paramount Bed try to deliver the replacement parts quickly after your order, but sometimes it might take a time for delivery due to distribution conditions. Please check the delivery time when ordering.
- When you replace the parts, Paramount Bed recommend to replace fixing parts together with the replacing parts (some parts are not suitable for reuse).
- When replacing the parts, please make sure they are fixed by the screws, bolts and nuts.



Caution

- When fixing, please do not apply too much pressure on the screws and bolts. Too much tightening may cause deformation or damage.
- Be careful handling snap-pins. When you attach or detach, there is a possibility that snap-pin may pop out in unexpected direction and cause an injury.


1. MATTRESS BASE • SIDE RAIL (TRANSFER BOARD)

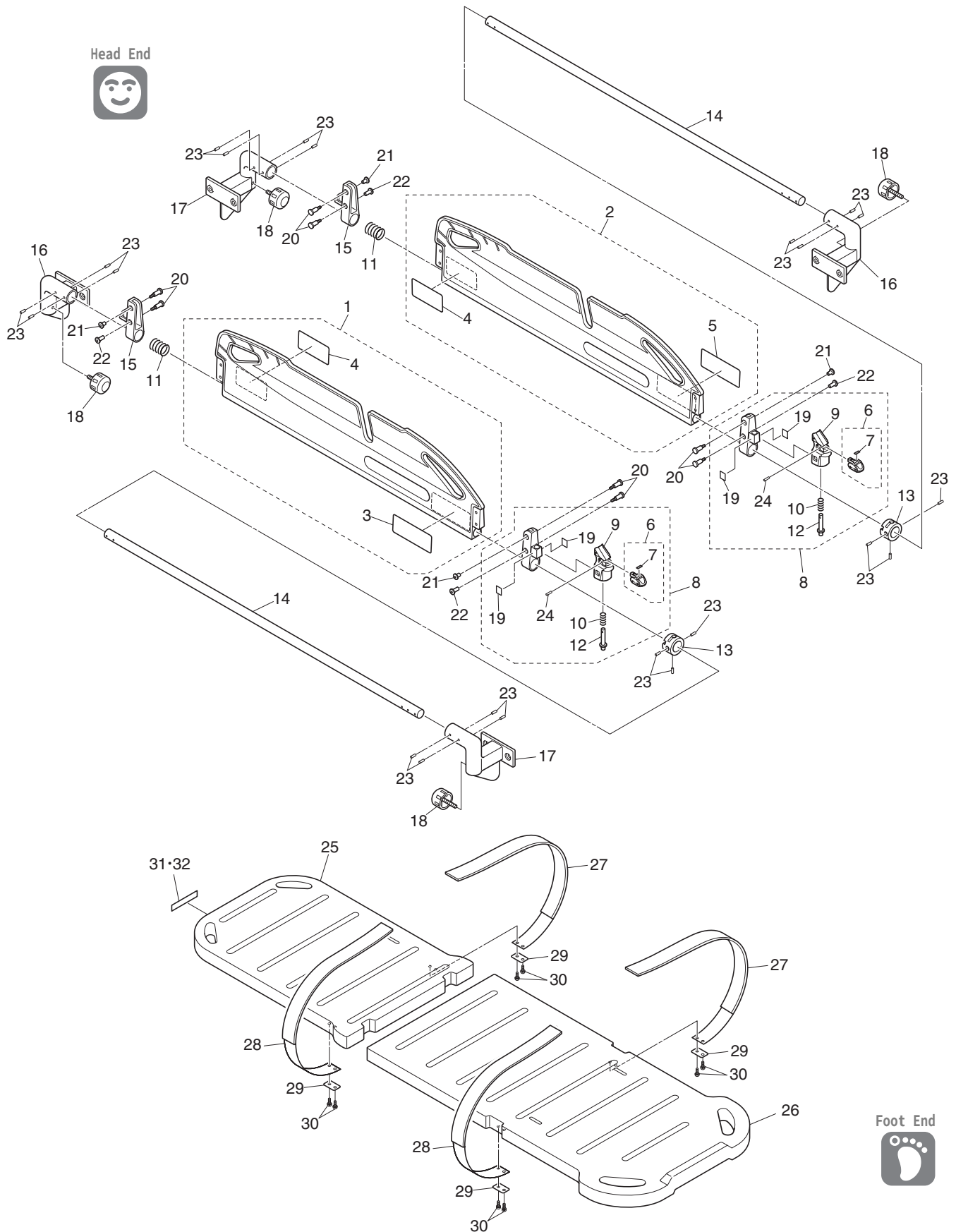

NO.	Parts NO.	Parts Name	Required Per Unit	Remarks
1	3F07161600A0	Side Rail R AS	1	Consist of NO.3,4
2	3F07161700A0	Side Rail L AS	1	Consist of NO.4,5
3	3F06857700A0	Warning Label R*	1	
4	3F06919600A0	Warning Caution Label*	2	
5	3F07006900A0	Warning Label L*	1	
6	3F07104500A0	W locking Lever	2	Consist of NO.7
7	3F07004900A0	Red Label*	2	
8	3F06999400A0	Lock Device AS	2	Consist of NO.6,7,9,10,12,19,24.
9	3F06833600A0	Locking Lever	2	
10	3F06942800A0	Lever Spring	2	
11	3F06806300A0	Spring For Side Rail	2	
12	3F06806000A2	Plunger Pin	2	
13	3F06833400A0	Plunger Holder	2	
14	3F06942500A0	Side Rail Pipe	2	
15	3F06942600A0	Stopper Head End	2	
16	3K00927401A0	Side Rail Holder R	2	
17	3K00927402A0	Side Rail Holder L	2	
18	3F00028700E0	Knob Screw: Type B	4	Light blue color
19	3F06966100A0	Two-Color Label *	4	
20	3K01013000A0	Screw: JCB6 × 23A	8	
21	001040406012	Nut: JCN6 × 12A	4	
22	001040406017	Nut: JCN6 × 17A	4	
23	005100S04812	BR4.8 × 11.9*	22	
24	004000604016	SP4 × 16*	2	
25	3K00926500B0	Mattress Base: HEAD	1	
26	3K00926600B0	Mattress Base: FOOT	1	
27	3K00926700A0	Restraint Belt A	2	
28	3K00926800A0	Restraint Belt B	2	
29	3K00630100B0	Fixing Metal (Restraint Belt)	4	
30	000111305010	Screw b5 × 10S*	8	
31	3F07006500A0	Label for I.V. Storage	1	For KK-726, KK-728E
32	3K00926900A0	Label for I.V. Storage	1	For KK-725, KK-727

The parts come with "" mark after parts name, are not suitable for reuse and could be worn out due to long-term usage. Please check the conditions when maintaing.


2. MAIN FRAME • HI-LO LINK

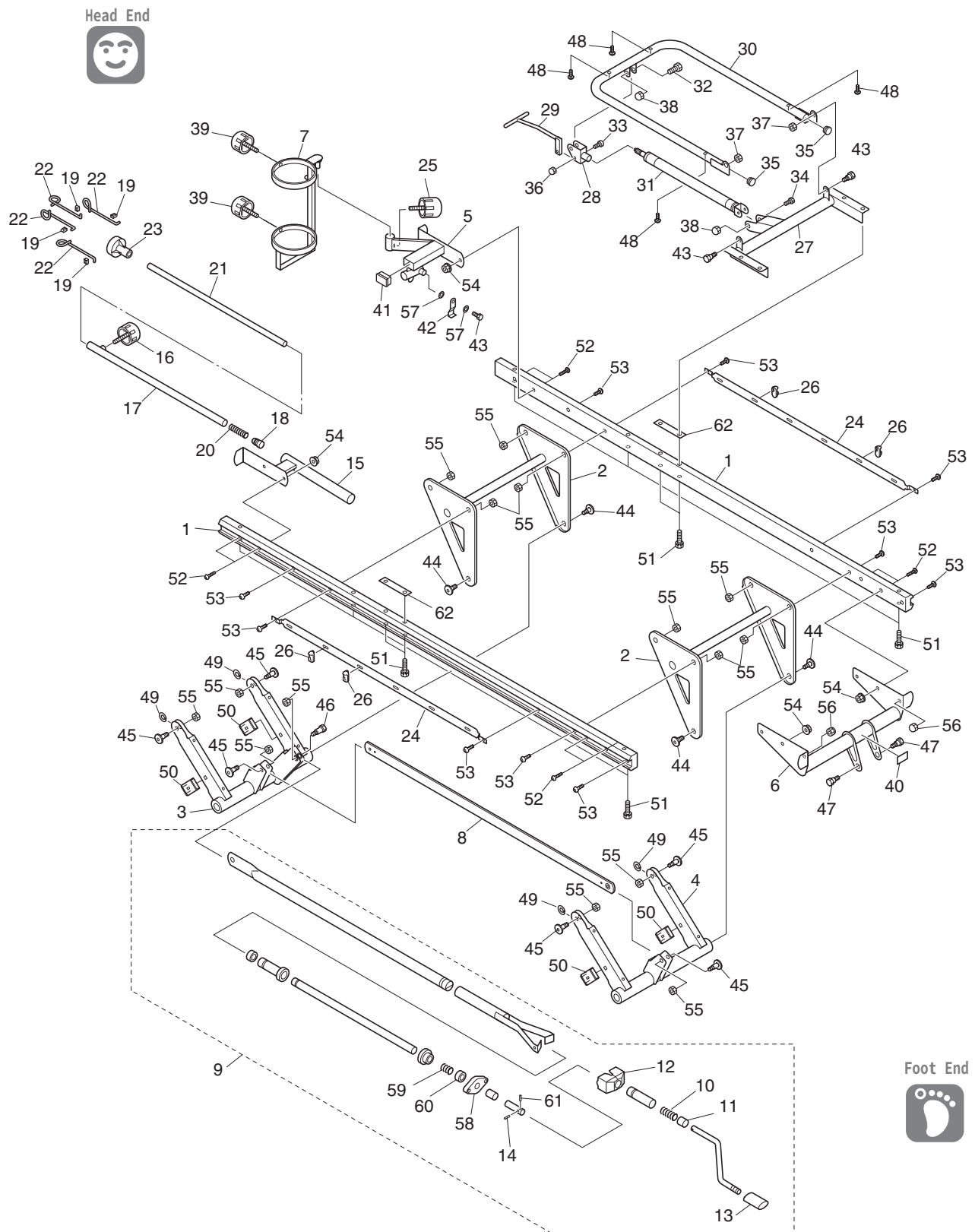

NO.	Parts NO.	Parts Name	Required Per Unit	Remarks
1	3F06854700A0	Main Frame: LONG SIDE	2	
2	3F07083200A0	Frame Holder	2	
3	3F07161000A0	HL Link AS: Head End	1	
4	3F07161100A0	HL Link AS: Foot End	1	
5	3F07161200A0	Oxygen Holder Receptacle AS	1	
6	3F07161300A0	Shaft Unit Link	1	Consist of NO.40 with product statement label
7	3F07161400A0	Oxygen Holder AS	1	
8	3F06802620A0	Hi-Lo Linkage Shaft	1	
9	3F07160900A0	Hi-Lo Crank Handle AS	1	Consist of NO.10,11,12, 13,14,58,59,60,61
10	3F00020500A0	Spring For Handle	1	
11	3F00047200B0	C Handle Collar	1	
12	3F00140700B0	Boss Cover	1	
13	3F00140800B0	Grip*	1	
14	004000604020	SP4 x 20*	1	
15	3K00936020A0	I.V. Holder AS	1	
16	3F00028700D0	Bolt Type B	1	Gray color
17	3F01628000A0	Outer Layer	1	
18	3F00034600A0	Base Cap*	1	
19	3F00057600B0	Hook Stoppers	4	
20	3F00057700A0	Stopper Spring	1	
21	3F01628200A0	Inner Layer	1	
22	3F01628300A0	Hook	4	
23	3F01628400B0	Hanger Top	1	
24	3F06891500A0	Accessory Rail	2	
25	3F00028700E0	Bolt Type B	1	Light blue color
26	3K00335500A0	Hook For Urine Collection Bag	4	
27	3F06949220A0	Back Frame Holder AS	1	*Only KK-726, KK-728E
28	3K00641400A0	GS Adopter AS	1	*Only KK-726, KK-728E
29	3K00932800A0	Back Raise Lever AS	1	*Only KK-726, KK-728E
30	3K00933220A0	Back Frame AS	1	*Only KK-726, KK-728E
31	3F01427100A0	GS30	1	*Only KK-726, KK-728E
32	3K00938200A0	Screw: B8 x 45H	1	*Only KK-726, KK-728E
33	3K01078100A0	Screw b6 x 34*	1	*Only KK-726, KK-728E
34	3K01205500A1	Screw: B8 x 52	1	*Only KK-726, KK-728E
35	210003000004	Driving Cap22*	2	*Only KK-726, KK-728E
36	001002306000	Nut: N60*	1	*Only KK-726, KK-728E
37	001010308000	Nut: UN8*	2	*Only KK-726, KK-728E
38	001012308000	Nut: UN80*	2	*Only KK-726, KK-728E
39	3F06891600A0	Bolt: Type B	2	Light blue color
40	3F00027800A0	Hi-Lo Label R*	1	
41	210025000001	Driving Cap 40 x 25*	1	
42	3K00938900A0	Rotation Stopper Fitting	1	
43	3F00022100A0	Screw B8M10	3	*KK-726, KK-727 takes 1
44	3K00715700A0	Shoulder Screw B8M10*	4	
45	3K00715800A0	Shoulder Screw B14M10*	6	
46	3K00805600A0	Screw: B25M10*	1	
47	3K00818600A0	Screw: B9M10D*	2	
48	3K00938500A0	Screw: B8 x 30*	4	*Only KK-726, KK-728E
49	3K01050400A0	Noise Reduction Washer	4	
50	3K01804300A1	Plastic Frame	4	
51	000010308065	Screw: B8 x 65SW*	10	*KK-726, KK-728 takes 6
52	000072308050	Screw: B8 x 50FP*	8	
53	000022310040	Screw: B10 x 40*	10	
54	001070108000	Nut: UN8FP*	6	
55	001010310000	Nut: UN10*	15	
56	001012310000	Nut: UN100*	2	
57	3K01011000A0	Plastic Washer	2	
58	3K00752000A0	Shaft Holder	1	
59	3K00615000A0	Bushing: Spring Clutch	1	
60	3K00614900A0	Inner Screw	1	
61	004000605025	SP 5 x 25*	1	
62	3K01010500A0	Spacer	2	*Only KK-726, KK-727

The parts with "" mark after parts name, are not suitable for reuse and could be worn out due to long-term usage. Please check the conditions when maintaining.

3. BASE FRAME

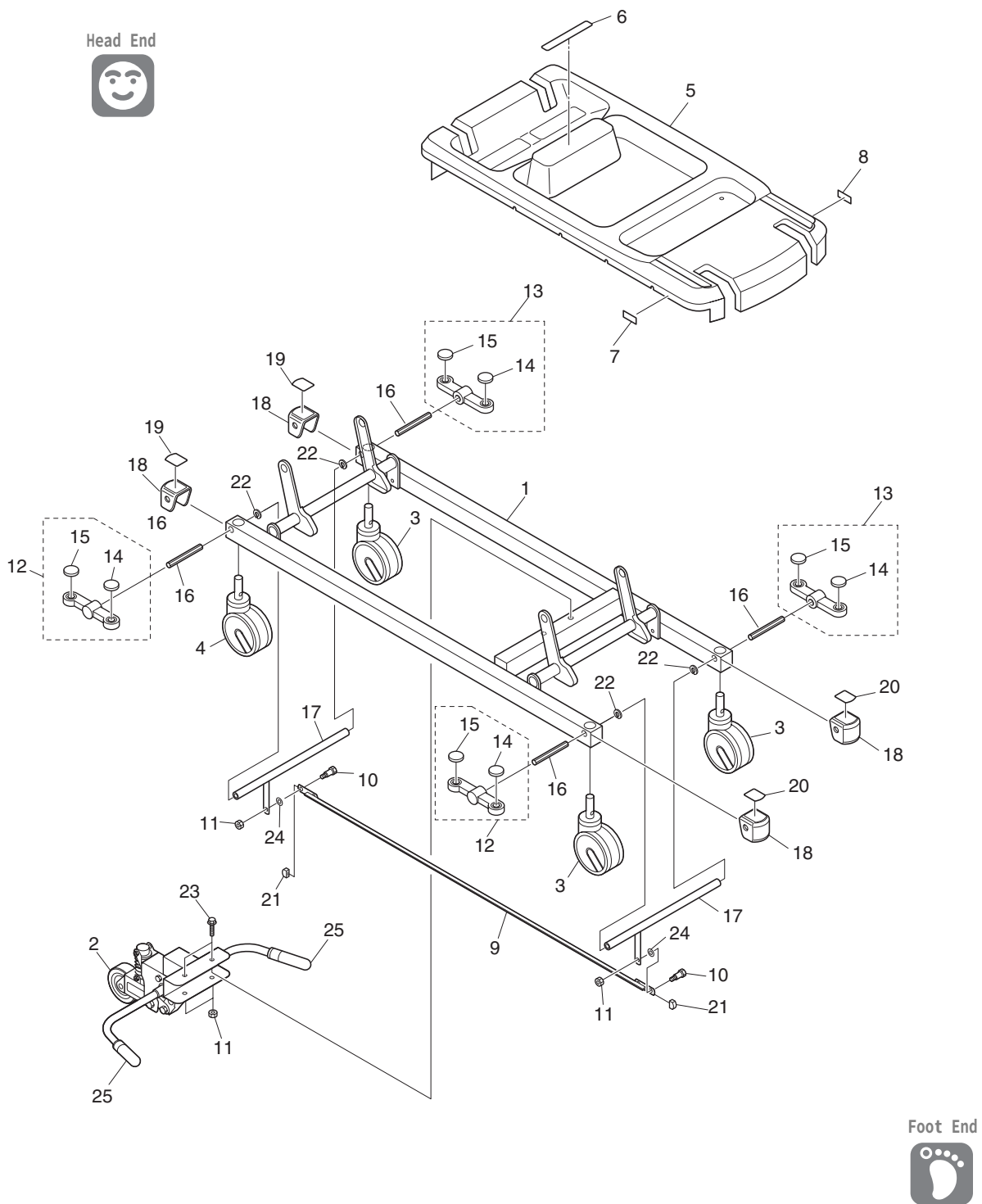

NO.	Parts NO.	Parts Name	Required Per Unit	Remarks
1	3F07083400A0	Base Frame AS	1	
2	3F06804800A0	Center Wheel AS	1	
3	T150E8NS2	Caster	3	
4	T150E8NS2X	Caster	1	*Type of transmission
5	3F07161500A0	Tray	1	Consist of NO.6,7,8.
6	3F06982800A0	Caution Label*	1	
7	3F07021300A0	Operating Label R*	1	
8	3F07021400A0	Operating Label L*	1	
9	3F06893720A0	CL interlocking AS	1	
10	3F00022100A0	Screw B5 M8	2	
11	001010308000	Nut: UN8*	4	
12	3F06807200A0	Pedal R AS	2	
13	3F06807300A0	Pedal L AS	2	
14	200611000003	Pedal Rubber (Red)	4	
15	200611000004	Pedal Rubber (Green)	4	
16	3F06806100A0	Cam Shaft	4	
17	3F06805400A0	Cam Shaft Holder AS	2	
18	3F06806900A0	Base Frame Cap	4	
19	3F07006600A0	Operating Label: Head*	2	
20	3F07006800A0	Operating Label: Foot*	2	
21	210020000002	Driving Cap 22 x10*	2	
22	3K01804400A0	Plastic Washer Type C	4	
23	3K01010400A1	Screw: B8 x 67F	2	
24	3K01011000A0	Plastic Washer	2	
25	041200000007	Grip	2	

The parts come with "" mark after parts name, are not suitable for reuse and could be worn out due to long-term usage. Please check the conditions when maintaing.

PARAMOUNT BED